


## HemePath Consult (w/CBC)

Order Name: **HemePath Consult**  
Test Number: 2904600  
Revision Date: 11/16/2023

TEST NAME	METHODOLOGY	LOINC CODE
Complete Blood Count (CBC) with Automated Differential	Flow cytometry	See Individual Assays
Reticulocyte (Retic) Count	Flow cytometry	
Immature Platelet Fraction	Flow cytometry	71693-6
HemePath Consult order		

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	5 mL (1 mL)	Whole Blood	EDTA (Lavender Top)	Refrigerated
Alternate 1	1 mL (0.5 mL)	Whole Blood	EDTA (Lavender) Microtainer/Bullet	Refrigerated
Instructions	<p><b>For best results: Room temperature specimens should be tested within 24hrs, otherwise send Refrigerated.</b></p> <p>Refrigerated specimens can be tested up to 48hrs. Specimens received after 24hrs will not receive a manual differential.</p> <p>Slides must be made within 24hr of collection.</p> <p><b>Specimens received greater than 48hrs old will be canceled.</b></p> <p><i>Note: This test cannot be added on to an A1c that has already been performed.</i></p>			

GENERAL INFORMATION	
Testing Schedule	Daily
Expected TAT	1-2 Days
Clinical Use	Pathology Report will be issued on this on Order/Result Code: <b>[8077777] Hematology Other Report</b> , at an additional charge to the patient.
Notes	Provide patient history as available. Testing includes CBC with automated differential, IPF, Retic Count and pathologist interpretation. If this testing is performed at your laboratory please send these results with the smears and the lavender tube. If the question is anemia, consider ordering an Anemia Analyzer with the algorithmic reflex ordering of the appropriate chemistry tests.
CPT Code(s)	85025, 85045, 85055, Additional professional code determined with clinical findings
Service Provided By	 <b>labcorp</b> Oklahoma, Inc.