


Prostate Specific Antigen (PSA), Free and Total

Order Name: **PSA FREE**
Test Number: 2001575
Revision Date: 10/28/2025

TEST NAME	METHODOLOGY	LOINC CODE
Free PSA	Chemiluminescence Assay	10886-0
Total PSA	Chemiluminescence Assay	2857-1
% Free PSA	Calculation	12841-3

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (mL)	Specimen Type	Specimen Container	Transport Environment
Preferred	2 mL (1)	Serum	Clot Activator SST	Frozen
Instructions	Collect prior to biopsy, prostatectomy, prostatic massage or digital rectal exam. Allow 30min. for specimen to clot. Separate Serum from cells/gel within 3 hours of collection. Freeze if testing will not be completed within 24 hours. Specimen stability: Room temperature N/A; Refrigerated 24hrs; Frozen 1mo.			

GENERAL INFORMATION	
Testing Schedule	Mon - Fri
Expected TAT	2-4 Days
Notes	Testing performed via Abbott Architect Immunoassay method. Results obtained with different assay methods should not be used interchangeably in serial testing. It is recommended that only one assay method be used consistently to monitor each patient's course of therapy.
CPT Code(s)	84153, 84154
Service Provided By	 labcorp Oklahoma, Inc.