

## Anti-Nuclear Antibody Screen (ANA) with Reflex to Titer

Order Name: **ANA SCR**  
 Test Number: 5500050  
 Revision Date: 11/12/2020

TEST NAME	METHODOLOGY	LOINC CODE
Anti-Nuclear Antibody Screen (ANA) with Reflex to Titer	<u>Indirect Fluorescent Antibody</u>	8061-4

### SPECIMEN REQUIREMENTS

Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL	Serum	Clot Activator (SST or Red No-Gel)	Refrigerated
Instructions	Allow specimen to clot completely at room temperature. <b>Separate serum from cells ASAP or within 2 hours of collection.</b> Stability After separation from cells: Ambient 8 hours, Refrigerated 7 days, Frozen 3 months (avoid repeated freeze/thaw cycles).			

### GENERAL INFORMATION

Testing Schedule	Mon - Fri
Expected TAT	3 Days
Clinical Use	To aid in the diagnose autoimmune diseases.
Notes	If ANA screen is positive >1:80; titer is automatically performed. ANA screening is done by IFA using HEp-2 Cells. See <a href="https://www.anapatterns.org/index.php">https://www.anapatterns.org/index.php</a> for current clinical associations and identified patterns.

REFLEX ORDER CODE	REFLEX TEST NAME	RESULT CODE	RESULT LONG NAME
5512450	Antinuclear Antibody Titer	5500100	Antinuclear Antibody Titer
		5500150	Antinuclear Antibody Pattern
		5577550	Antinuclear Antibody Titer 2
		5577650	Antinuclear Antibody Pattern 2
		5577600	Antinuclear Antibody Titer 3
		5577700	Antinuclear Antibody Pattern 3

CPT Code(s)	86038 (86039 will bill in addition if reflexed to titer)
-------------	--

Service Provided By

