


**Ethanol Screen**

Order Name: **ETHANOL SC**  
Test Number: 4301875  
Revision Date: 05/20/2021

TEST NAME	METHODOLOGY	LOINC CODE
Ethanol Screen	Enzymatic .....	20470-1

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL (0.5)	Plasma	Lithium Heparin PST (Green/Gray Top)	Refrigerated
Alternate 1	1 mL (0.5)	Serum	Clot Activator SST	Refrigerated
Instructions	Specimen stability: Ambient 2 weeks, Refrigerated 6 months, Frozen 6 months Keep tightly stoppered - Do not remove stopper of collection tube. Use betadine or phisohex prep sponge. Do not use alcohol.			

GENERAL INFORMATION	
Testing Schedule	Daily
Expected TAT	1-2 days
Clinical Use	Useful in detection of ethanol (alcohol) in blood. Qualitative test.
CPT Code(s)	80307
Service Provided By	 <b>labcorp</b> Oklahoma, Inc.