

Order Name: GLUCAGON Test Number: 2007000 Revision Date: 12/12/2022

Instructions Enzyme-Linked Immunosorbent Assay (ELISA) 2338-2 SPECIMEN REQUIREMENTS Specimen Volume (min) Specimen Type Specimen Container Transport Environment Preferred 1mL (0.5) Plasma EDTA (Lavender Top) Frozen Instructions Notes: 0.5 mL (Note: This volume Does NOT allow for repeat testing.) Specimen Type: Lavender-top (EDTA) tube, (chilled) Specimen Storage: FREEZE Specimen Collection: Separate plasma and transfer specimen to a plastic transport tube before freezing. To avoid delays in turnaround time when requesting multiple tests on frozen samples, please submit separate frozen specimens for each test requested. Special Instructions: This test may exhibit interference when sample is collected from a person who is consuming a supplement with a high dose of biotin (also termed as vitamin B7 or B8, vitamin H, or coenzyme R). It is recommended to ask all patients who may be indicated for this test about biotin supplementation. Patients should be cautioned to stop biotin consumption at least 72 hours prior to the collection of a sample. Cause for Rejection: Lipemia, hemolysis or icterus; non EDTA plasma sample received; non?frozen sample received Specimen Stability: Ambient: Unstable, Refrigerated : Unstable, Frozen: 7 days	TEST NAME			METHODOLOGY	LOINC CODE	
SPECIMEN REQUIREMENTS Specimen Specimen Volume (min) Specimen Type Specimen Container Transport Environment Preferred 1mL (0.5) Plasma EDTA (Lavender Top) Frozen Instructions Notes: 0.5 mL (Note: This volume Does NOT allow for repeat testing.) Specimen Type: Lavender-top (EDTA) tube, (chilled) Specimen Storage: FREEZE Specimen Collection: Separate plasma and transfer specimen to a plastic transport tube before freezing. To avoid delays in turnaround time when requesting multiple tests on frozen samples, please submit separate frozen specimens for each test requested. Special Instructions: This test may exhibit interference when sample is collected from a person who is consuming a supplement with a high dose of biotin (also termed as vitamin B7 or B8, vitamin H, or coenzyme R). It is recommended to ask all patients who may be indicated for this test about biotin supplementation. Patients should be cautioned to stop biotin consumption at least 72 hours prior to the collection of a sample. Cause for Rejection: Lipemia, hemolysis or icterus; non EDTA plasma sample received; non?frozen sample received	TESTINAME					
Specimen Specimen Volume (min) Specimen Type Specimen Container Transport Environment Preferred 1mL (0.5) Plasma EDTA (Lavender Top) Frozen Instructions Notes: 0.5 mL (Note: This volume Does NOT allow for repeat testing.) Specimen Type: Lavender-top (EDTA) tube, (chilled) Specimen Storage: FREEZE Specimen Collection: Separate plasma and transfer specimen to a plastic transport tube before freezing. To avoid delays in turnaround time when requesting multiple tests on frozen samples, please submit separate frozen specimens for each test requested. Special Instructions: This test may exhibit interference when sample is collected from a person who is consuming a supplement with a high dose of biotin (also termed as vitamin B7 or B8, vitamin H, or coenzyme R). It is recommended to ask all patients who may be indicated for this test about biotin supplementation. Patients should be cautioned to stop biotin consumption at least 72 hours prior to the collection of a sample. Cause for Rejection: Lipemia, hemolysis or icterus; non EDTA plasma sample received; non?frozen sample received;	Glucagon Level			Enzyme-Linked Immunosorbent Assay (El	LISA) 2338-2	
Specimen Specimen Volume (min) Specimen Type Specimen Container Transport Environment Preferred 1mL (0.5) Plasma EDTA (Lavender Top) Frozen Instructions Notes: 0.5 mL (Note: This volume Does NOT allow for repeat testing.) Specimen Type: Lavender-top (EDTA) tube, (chilled) Specimen Storage: FREEZE Specimen Collection: Separate plasma and transfer specimen to a plastic transport tube before freezing. To avoid delays in turnaround time when requesting multiple tests on frozen samples, please submit separate frozen specimens for each test requested. Special Instructions: This test may exhibit interference when sample is collected from a person who is consuming a supplement with a high dose of biotin (also termed as vitamin B7 or B8, vitamin H, or coenzyme R). It is recommended to ask all patients who may be indicated for this test about biotin supplementation. Patients should be cautioned to stop biotin consumption at least 72 hours prior to the collection of a sample. Cause for Rejection: Lipemia, hemolysis or icterus; non EDTA plasma sample received; non?frozen sample received;						
Preferred1mL (0.5)PlasmaEDTA (Lavender Top)FrozenInstructionsNotes: 0.5 mL (Note: This volume Does NOT allow for repeat testing.) Specimen Type: Lavender-top (EDTA) tube, (chilled) Specimen Storage: FREEZE Specimen Collection: Separate plasma and transfer specimen to a plastic transport tube before freezing. To avoid delays in turnaround time when requesting multiple tests on frozen samples, please submit separate frozen specimens for each test requested. Special Instructions: This test may exhibit interference when sample is collected from a person who is consuming a supplement with a high dose of biotin (also termed as vitamin B7 or B8, vitamin H, or coenzyme R). It is recommended to ask all patients who may be indicated for this test about biotin supplementation. Patients should be cautioned to stop biotin consumption at least 72 hours prior to the collection of a sample. Cause for Rejection: Lipemia, hemolysis or icterus; non EDTA plasma sample received; non?frozen sample received	SPECIMEN REQUI	REMENTS				
Instructions Notes: 0.5 mL (Note: This volume Does NOT allow for repeat testing.) Specimen Type: Lavender-top (EDTA) tube, (chilled) Specimen Storage: FREEZE Specimen Collection: Separate plasma and transfer specimen to a plastic transport tube before freezing. To avoid delays in turnaround time when requesting multiple tests on frozen samples, please submit separate frozen specimens for each test requested. Special Instructions: This test may exhibit interference when sample is collected from a person who is consuming a supplement with a high dose of biotin (also termed as vitamin B7 or B8, vitamin H, or coenzyme R). It is recommended to ask all patients who may be indicated for this test about biotin supplementation. Patients should be cautioned to stop biotin consumption at least 72 hours prior to the collection of a sample. Cause for Rejection: Lipemia, hemolysis or icterus; non EDTA plasma sample received; non?frozen sample received	Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment	
 Specimen Type: Lavender-top (EDTA) tube, (chilled) Specimen Storage: FREEZE Specimen Collection: Separate plasma and transfer specimen to a plastic transport tube before freezing. To avoid delays in turnaround time when requesting multiple tests on frozen samples, please submit separate frozen specimens for each test requested. Special Instructions: This test may exhibit interference when sample is collected from a person who is consuming a supplement with a high dose of biotin (also termed as vitamin B7 or B8, vitamin H, or coenzyme R). It is recommended to ask all patients who may be indicated for this test about biotin supplementation. Patients should be cautioned to stop biotin consumption at least 72 hours prior to the collection of a sample. Cause for Rejection: Lipemia, hemolysis or icterus; non EDTA plasma sample received; non?frozen sample received 	Preferred	1mL (0.5)	Plasma	EDTA (Lavender Top)	Frozen	
 Specimen Storage: FREEZE Specimen Collection: Separate plasma and transfer specimen to a plastic transport tube before freezing. To avoid delays in turnaround time when requesting multiple tests on frozen samples, please submit separate frozen specimens for each test requested. Special Instructions: This test may exhibit interference when sample is collected from a person who is consuming a supplement with a high dose of biotin (also termed as vitamin B7 or B8, vitamin H, or coenzyme R). It is recommended to ask all patients who may be indicated for this test about biotin supplementation. Patients should be cautioned to stop biotin consumption at least 72 hours prior to the collection of a sample. Cause for Rejection: Lipemia, hemolysis or icterus; non EDTA plasma sample received; non?frozen sample received 	Instructions	,		ut testing.)		
Specimen Collection: Separate plasma and transfer specimen to a plastic transport tube before freezing. To avoid delays in turnaround time when requesting multiple tests on frozen samples, please submit separate frozen specimens for each test requested. Special Instructions: This test may exhibit interference when sample is collected from a person who is consuming a supplement with a high dose of biotin (also termed as vitamin B7 or B8, vitamin H, or coenzyme R). It is recommended to ask all patients who may be indicated for this test about biotin supplementation. Patients should be cautioned to stop biotin consumption at least 72 hours prior to the collection of a sample. Cause for Rejection: Lipemia, hemolysis or icterus; non EDTA plasma sample received; non?frozen sample received		1 11				
requesting multiple tests on frozen samples, please submit separate frozen specimens for each test requested. Special Instructions: This test may exhibit interference when sample is collected from a person who is consuming a supplement with a high dose of biotin (also termed as vitamin B7 or B8, vitamin H, or coenzyme R). It is recommended to ask all patients who may be indicated for this test about biotin supplementation. Patients should be cautioned to stop biotin consumption at least 72 hours prior to the collection of a sample. Cause for Rejection: Lipemia, hemolysis or icterus; non EDTA plasma sample received; non?frozen sample received		1 0		nen to a plastic transport tube before freezing	1 To avoid delays in turnaround time when	
biotin (also termed as vitamin B7 or B8, vitamin H, or coenzyme R). It is recommended to ask all patients who may be indicated for this test about biotin supplementation. Patients should be cautioned to stop biotin consumption at least 72 hours prior to the collection of a sample. Cause for Rejection: Lipemia, hemolysis or icterus; non EDTA plasma sample received; non?frozen sample received						
biotin supplementation. Patients should be cautioned to stop biotin consumption at least 72 hours prior to the collection of a sample. Cause for Rejection: Lipemia, hemolysis or icterus; non EDTA plasma sample received; non?frozen sample received		Special Instructions: This test may exhibit interference when sample is collected from a person who is consuming a supplement with a high dose of				
Cause for Rejection: Lipemia, hemolysis or icterus; non EDTA plasma sample received; non?frozen sample received		biotin (also termed as vitamin !	B7 or B8, vitamin H, or coenzy	me R). It is recommended to ask all patients v	who may be indicated for this test about	
			'	1 1	I	
Specimen Stability: Ambient: Unstable, Refrigerated : Unstable, Frozen: 7 days					ple received	
		Specimen Stability: Ambient:	. Unstable, Refrigerated : Unsta	able, Frozen: 7 days		
	GENERAL INFORM					

GENERAL INFORMATION	
Expected TAT	3 - 10 days
Notes	Labcorp Test Code: 004622
CPT Code(s)	82943
Service Provided By	Oklahoma, Inc.