

## Chlamydia Species Antibody Panel, IgM, IgG, IgA

Order Name: **LGV AB**  
 Test Number: 5585525  
 Revision Date: 10/12/2017

TEST NAME	METHODOLOGY	LOINC CODE
Chlamydia Species Antibody Panel, IgM, IgG, IgA	<a href="#">Micro-immunofluorescence</a>	


### SPECIMEN REQUIREMENTS

Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL (0.2 mL)	Serum	Clot Activator SST	Room Temperature

**Instructions**

**Specimen Type:** Serum Separator (Gel) Tube  
**Specimen Storage:** Room temperature Stability: Room Temperature: 7 days Refrigerated: 14 days Frozen: 30 days  
**Specimen Collection:** Serum must be separated from cells within 45 minutes of venipuncture. Send serum in a plastic transfer tube.  
**Special Instructions:** Testing referred to Quest Diagnostics Infectious Disease QDICA#19553

### GENERAL INFORMATION

<b>Clinical Use</b>	Lymphogranuloma Venereum Panel Includes: Chlamydia trachomatis (IgG, IgM, IgA), Chlamydomphila pneumoniae (IgG, IgM, IgA), Chlamydomphila psittaci (IgG, IgM, IgA)
<b>Notes</b>	Labcorp Test Code: 824784
<b>Service Provided By</b>	 <b>labcorp</b> Oklahoma, Inc.