

**Infliximab Activity and Neutralizing Antibody**

Order Name: **Infliximab NAB**  
Test Number: 6905051  
Revision Date: 01/02/2020

TEST NAME	METHODOLOGY	LOINC CODE
Infliximab Activity and Neutralizing Antibody	Electrochemiluminescent Immunoassay	

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	2 mL (1 mL)	Serum	Clot Activator SST	Frozen
Instructions	<p><b>Notes:</b> 1 mL (Note: This volume Does NOT allow for repeat testing).</p> <p><b>Specimen Type:</b> Red-top tube or gel-barrier tube</p> <p><b>Specimen Storage:</b> Frozen (preferred) or refrigerated Stability: Room temperature: 7 days Refrigerated: 7 days Frozen: 359 days Freeze/thaw cycle: stable x6</p> <p><b>Specimen Collection:</b> Serum must be separated from cells within 45 minutes of venipuncture. Send serum in a plastic transport tube. To avoid delays in turnaround time when requesting multiple test on frozen samples, PLEASE SUBMIT SEPARATE FROZEN SPECIMENS FOR EACH TEST REQUESTED.</p> <p><b>Special Instructions:</b> Testing is performed at Esoterix Endocrinology Laboratory: ES#803870 Test includes Serial Monitoring.</p>			

GENERAL INFORMATION	
Expected TAT	5-10 days
Notes	Labcorp Test Code: 503870
CPT Code(s)	80230, 82397
Service Provided By	 <b>labcorp</b> Oklahoma, Inc.