

Chlamydia trachomatis Urine - TMA

Order Name: **CT Urine**
Test Number: **6910111**
Revision Date: **07/10/2017**

TEST NAME	METHODOLOGY	LOINC CODE
Chlamydia trachomatis Urine	Transcription-Mediated Amplification	42931-6

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (mL)	Specimen Type	Specimen Container	Transport Environment
Preferred	2mL	Urine, First Void	APTIMA Urine Transport Tube	Refrigerated
Instructions	First catch urine. Patient should not have urinated for at least 1 hr prior to collection. Female patients should not cleanse the labial area prior to collection. Transfer 2 mL urine to APTIMA Urine Specimen Transport Tube. Liquid level must be between fill lines on tube. Stability: Refrigerated = 30 days.			
Please contact your account rep if training and APTIMA Urine Specimen Transport supplies have not been supplied to your practice.				

GENERAL INFORMATION	
Testing Schedule	Mon-Fri
Expected TAT	1-2 days after set-up
CPT Code(s)	87491
Service Provided By	 labcorp Oklahoma, Inc.