

Order Name: **T Helper**Test Number: 3654600
Revision Date: 05/24/2023

TEST NAME		METH	IODOLOGY	LOINC CODE	
Lymphocyte Absolute CD4 Count		Flow	Flow cytometry		
SPECIMEN REQUIREM	IENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment	
Preferred	5 mL each (3 mL each)	Whole Blood	EDTA (Lavender) and ACD Sol A/B (Yellow Top)	Room Temperature	
Instructions	Collect Whole blood in Both ACD Solution A or B (Yellow Top) and EDTA (Lavender Top). Keep whole blood, do not centrifuge or refrigerate. Deliver to the laboratory ASAP - Same Day as Collection! Collect Monday through Wednesday only. Specimens must be received at Labcorp Oklahoma Inc. by Wednesday afternoon or collect on Monday. Stability: Room Temperature: Yellow-top (ACD-A) or (ACD-B) tube: Lavender-top (EDTA) tube: 1 day				

GENERAL INFORMATION		
Testing Schedule	Testing schedule may vary.	
Expected TAT	3-5 days	
Clinical Use	Marker for level of immunocompetence. The T4 Lymphocytes reports out values for Lymphocyte Percent CD4 and Lymphocyte Absolute CD4	
CPT Code(s)	86361	
Service Provided By	labcorp Oklahoma, Inc.	