

## ThyGenNEXT with Reflex to ThyraMIR

Order Name: ThyGenNEXT Rflx

Test Number: 6907017 Revision Date: 10/01/2022

METHODOLOGY	LOINC CODE
MicroRNA (miRNA) Profiling	n/a
MicroRNA (miRNA) Profiling	n/a
Specimen Information	n/a
Patient Information	n/a
Calculation	n/a
Interpretive information.	n/a
	MicroRNA (miRNA) Profiling  MicroRNA (miRNA) Profiling  Specimen Information  Patient Information  Calculation

Thy Gen interpreta	illon		terpretive information.	Tiva		
SPECIMEN REQUIREMENTS						
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment		
Preferred	1 Vial	Fine Needle Aspirate	RNARetain vial	Room Temperature		
Instructions						
	Contact Labcorp Oklahoma, Inc. for FNA collection kit with special collection vial, instructions and requisition  Specimen collection: Fine needle aspiration of thyroid nodule in RNARetain vial (small amount, 3rd or 4th pass)					
	Specimen preparation: Send vial with cytology preparations at room temperature					
	Stability: Ambient - 6 weeks					
	Performing Lab Information	and Requisition				

GENERAL INFORMATION	
Testing Schedule	Varies
Expected TAT	7-14 days after pathology interpretation
Clinical Use	ThyGeNEXT® with Reflex to ThyraMIR® better discriminates benign from malignant nodules and provides risk assessment. TERT and BRAF mutations are included in ThyGeNEXT® . If mutations in ThyGeNEXT® are negative or not fully indicative of malignancy, the ThyraMIR® testing will be performed as reflex.
Notes	INTERFACE ORDERING: ThyGenNEXT Rflx Order should be placed in addition to the electronic Cytology order.  RESULT/REPORT CHANGE: ThyGenNEXT & ThyraMIR were previously reported as an addendum to the FNA Cytology report. They will now be reported from the General Laboratory section.
CPT Code(s)	0245U + 0018U if reflexed ThyraMIR Molecular testing is reflexed if mutations in ThyGeNEXT are negative or not fully indicative of malignancy.
Service Provided By	Oklahoma, Inc.