

Labcorp Oklahoma, Inc 9330 E 41st Street Tulsa, OK 74145 Order Name: SPERM P/OP
Test Number: 2929026
Revision Date: 10/15/2025

TEST NAME		METH	HODOLOGY	LOINC CODE
Semen Analysis Post Vasectomy		Macr	oscopic and Microscopic examination	
SPECIMEN REQUIREM	MENTS			
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	See Instructions	Semen	Sterile Screwtop Container	37'C (Body Temp)
Instructions	Must Call the Patient Service Center (918) 403-0225 to schedule specimen drop off! Specimens will NOT be accepted without an appointment. Volume: Entire ejaculate, Freshly collected sample, Specimen container: Clean, no additive container Provided by the Patient Service Center. Stability: Less than 2 hours from collection. Transport: Specimens should be sent to the laboratory at or near body temperatures (20°C to 37°C). Do not refrigerate or freeze. Note: Specimens should be submitted ASAP, Not exceeding 1 hour after collection. Download: Click Here to view Collection instructions.			

GENERAL INFORMATION		
Testing Schedule	Mon - Fri (09:00am -11:00am)	
Expected TAT	1-2 Days	
Clinical Use	To determine the presence or absence of sperm.	
Notes	Performed by Labcorp Oklahoma, Inc. References: World Health Organization. WHO Laboratory Manual for the Examination and Processing of Human Semen. 5th ed. Geneva, Switzerland: WHO; 2010.	
CPT Code(s)	83921, 89310	
Service Provided By	labcorp Oklahoma, Inc.	