


Creatinine, Urine Timed

Order Name: **CREAT TM U**
Test Number: 3006050
Revision Date: 05/03/2021

TEST NAME	METHODOLOGY	LOINC CODE
Total Urine Volume		3167-4
Creatinine, Urine, mg/dL	<u>Kinetic Alkaline Picrate (Jaffe)</u>	20624-3
Creatinine Urine Timed Estimated		
Creatinine, Urine, 24 Hour		2162-6
Hours Collected - Prompt		30211-7
Volume 24 Urine Prompt		

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL (0.5)	Urine, 24-hour	24 hour Urine Container	Refrigerated
Instructions	24 hour urine collection with no preservative. Record number of hours and volume in ML on the specimen container and any aliquots. Refrigerate urine during collection. 6 N HCL, Boric Acid and Sodium Carb are acceptable preservatives if collecting with another test that requires preservative. Specimen stability: Ambient 2 days, Refrigerated 6 days. Frozen 6 months			

GENERAL INFORMATION	
Testing Schedule	Mon - Fri
Expected TAT	1-2 days
Clinical Use	Useful as an aid in evaluating renal function.
CPT Code(s)	82570,(81050 if urine measured by RML)
Service Provided By	 labcorp Oklahoma, Inc.