


Uric Acid Urine Timed

Order Name: **URIC TM U**
Test Number: 3006175
Revision Date: 06/04/2021

TEST NAME	METHODOLOGY	LOINC CODE
Total Urine Volume		3167-4
Creatinine, Urine, mg/dL	<u>Kinetic Alkaline Picrate (Jaffe)</u>	20624-3
Creatinine Urine Timed Estimated		
Creatinine, Urine, 24 Hour		2162-6
Uric Acid 24 Hour Urine mg/dL	<u>Uricase</u>	21587-1
Uric Acid Urine Timed Estimated		
Uric Acid 24 Hour Urine mg/24hr		3087-4
Hours Collected - Prompt		30211-7
Volume 24 Urine Prompt		

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL (0.5)	Urine, 24-hour	24 hour Urine Container	Refrigerated
Instructions	Timed urine collection. No preservative. Record number of hours and volume in ml on the specimen container. Keep refrigerated. Specimen stability: Ambient 24 hours (4 days if pH adjusted >8.0 dropwise with NaOH). Refrigerated 7 days.			

GENERAL INFORMATION	
Testing Schedule	Daily
Expected TAT	1-2 days
Clinical Use	Useful for assessment and management of patients with kidney stones and gout.
CPT Code(s)	84560 (add 81050 if urine measured by RML)
Service Provided By	 labcorp Oklahoma, Inc.