

Estriol, Serum

Order Name: **ESTRIOL**
Test Number: 3801500
Revision Date: 12/12/2022

TEST NAME	METHODOLOGY	LOINC CODE
Estriol, Serum	<u>Immunochemiluminometric (ICMA)</u>	2251-7

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (mL)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL (0.3)	Serum	Clot Activator SST	Refrigerated
Instructions	<p>Notes: 0.3 mL (Note: This volume Does NOT allow for repeat testing.)</p> <p>Specimen Type: Red-top tube or gel-barrier tube</p> <p>Specimen Storage: Refrigerate</p> <p>Specimen Collection: If red-top tube is used, transfer separated serum to a plastic transport tube. All specimens should be drawn at the same time of day to allow comparison of values.</p> <p>Special Instructions: State weeks of gestation on the request form. Usually done as serial measurement.</p> <p>Specimen Stability: Ambient: 14 days, Refrigerated : 14 days, Frozen: 14 days</p>			

GENERAL INFORMATION	
Expected TAT	2 - 4 days
Notes	Labcorp Test Code: 004614
CPT Code(s)	82677
Service Provided By	 labcorp Oklahoma, Inc.