


## Vaginosis Profile Saline

Order Name: **VAG PROF**  
Test Number: 2915425  
Revision Date: 09/17/2018

TEST NAME	METHODOLOGY	LOINC CODE
Ph of vaginal discharge		48037-6
Whiff test	<a href="#">Amine</a>	46730-8
Gram stain		
Trichomonas Microscopic Examination	<a href="#">Microscopy</a>	32766-8
Vaginal Yeast Examination	<a href="#">Microscopy</a>	32765-0
Clue Cell Examination	<a href="#">Microscopy</a>	32764-3

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	See Instructions	Swab	Sterile Saline	Room Temperature
Instructions	<p>Obtain specimen from mucosal surface and place swab in saline tube.</p> <p>Samples contaminated with preparations containing iodine or by the immediate prior use of vaginal lubricants are not recommended.</p> <p>For optimal results, testing should be performed as soon as possible following collection; specimens must be &lt;24 hours old upon arrival.</p> <p>Note: The Vaginal Profile from Swab (with Trichomonas Antigen), is recommended for specimens being testing &gt;2 hours after collection. Please refer to test 2915445 for collection information.</p>			

GENERAL INFORMATION	
Testing Schedule	Daily
Expected TAT	1 Day
Clinical Use	Determines bacterial vaginosis or yeast vaginitis
CPT Code(s)	83986; 82120; 87210; 87205
Service Provided By	 <b>labcorp</b> Oklahoma, Inc.