

Digitoxin Level

Order Name: **DIGITOXIN**
Test Number: 4001800
Revision Date: 12/12/2022

TEST NAME	METHODOLOGY	LOINC CODE
Digitoxin Level	<u>Chemiluminescence Assay</u>	3559-2

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (mL)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL	Serum	Clot Activator (Red Top, No-Gel)	Refrigerated
Instructions	<p>Specimen Type: Red-top tube. Gel-barrier tubes are NOT recommended.</p> <p>Specimen Storage: SUBMISSION/TRANSPORT (<3 days): Room Temperature. For storage beyond three days, specimen should be refrigerated or frozen.</p> <p>Specimen Collection: Serum or plasma should be separated from cells within two hours of venipuncture. Submit serum or plasma in a plastic transport tube.</p> <p>Special Instructions: Testing performed at MEDTOX Laboratories, TC 3775</p> <p>Specimen Stability: Ambient: 3 days, Refrigerated : 7 days, Frozen: 2 months</p>			

GENERAL INFORMATION	
Expected TAT	2 - 5 days
Notes	Labcorp Test Code: 790440
CPT Code(s)	80375
Service Provided By	 labcorp Oklahoma, Inc.