


**Neisseria gonorrhoeae Swab - TMA**

Order Name: **GC Swab**  
Test Number: **6910115**  
Revision Date: 03/01/2021

TEST NAME	METHODOLOGY	LOINC CODE
Neisseria gonorrhoeae Swab	Transcription-Mediated Amplification	43305-2

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	Swab	Swab	APTIMA Multitest Swab kit	Ambient / Refrigerated
Alternate 1	Swab	Swab	APTIMA Unisex Swab Collection kit	Ambient / Refrigerated
Instructions	Vaginal swab in APTIMA Multitest swab collection kit, or endocervical or male urethral swab, rectal and throat specimens in APTIMA Unisex Swab Specimen Collection kit. Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube. Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A			

GENERAL INFORMATION	
Testing Schedule	Mon-Fri
Expected TAT	1-2 days after set-up
CPT Code(s)	87591
Service Provided By	 <b>labcorp</b> Oklahoma, Inc.