

Food Allergy Panel 2 IgG

Order Name: **A FOOD2 G**
Test Number: 3666126
Revision Date: 09/25/2017

TEST NAME	METHODOLOGY	LOINC CODE
Bakers Yeast Allergen IgG	ImmunoCAP	35538-8
Wheat IgG	ImmunoCAP	35537-0
Tomato Allergen IgG		35547-9
Soybean Allergen IgG		
Rye Food Allergen IgG	ImmunoCAP	51933-0
White Potato Allergen IgG		
Peanut Allergen IgG		
Orange Citrus Allergen IgG		
Oat allergen IgG	ImmunoCAP	51901-7
Malt Allergen IgG		
Egg White Allergen IgG		
Corn and Maize Allergen IgG		
Chocolate/Cacao IgG		
Chicken Meat Allergen IgG		
Caseine Allergen IgG		
Barley Allergen IgG	ImmunoCAP	51917-3
Pork Meat Allergen IgG		
Beef Allergen IgG		

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	9 mL (3.6 mL)	Serum	Clot Activator SST	Refrigerated
Instructions	Separate serum from cells ASAP or within 2 hours of collection. Transfer 9 mL(3.6 mL) serum to a Standard Transport Tube. Unacceptable: Hemolyzed, icteric, or lipemic specimens. Stability After separation from cells: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 year			

GENERAL INFORMATION

Testing Schedule	Assay Dependant
Expected TAT	1-4 Days
CPT Code(s)	86001x18
Service Provided By	 labcorp Oklahoma, Inc.