


**Candida and Trichomonas Vaginitis Swab**

Order Name: **CV-TV Swab**  
Test Number: 6987019  
Revision Date: 04/06/2020

TEST NAME	METHODOLOGY	LOINC CODE
Candida Species - Swab	<a href="#">Transcription-Mediated Amplification</a>	47000-5
Candida Glabrata - Swab	<a href="#">Transcription-Mediated Amplification</a>	n/a
Trichomonas Swab (Vaginal)	<a href="#">Transcription-Mediated Amplification</a>	70165-6

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	Swab	Swab	APTIMA Multitest Swab kit	Ambient / Refrigerated
Instructions	Collect Swab in APTIMA Multitest Swab collection kit. Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube. Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A			

GENERAL INFORMATION	
Testing Schedule	Mon-Fri
Expected TAT	1-2 days after set-up
CPT Code(s)	87481, 87661
Service Provided By	 <b>labcorp</b> Oklahoma, Inc.