

NMR LipoProfile

Order Name: **NMR LipoProfile**
Test Number: **5194452**
Revision Date: **03/31/2023**

TEST NAME	METHODOLOGY	LOINC CODE
NMR LipoProfile	Nuclear Magnetic Resonance (NMR)	

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (mL)	Specimen Type	Specimen Container	Transport Environment
Preferred	2 mL (1 mL)	Serum	Clot Activator (Red Top, No-Gel)	Refrigerated
Alternate 1	2 mL (1 mL)	Plasma	EDTA (Lavender Top)	Refrigerated
Alternate 2	2 mL (1 mL)	Plasma	Sodium Heparin (Green Top / No-Gel)	Refrigerated
Instructions	Specimen Type: Do Not use Gel separation tubes. Specimen Processing: Allow specimen to clot completely at room temperature if using red no gel. Separate serum or plasma from cells ASAP or within 2 hours of collection. Specimen Stability: After separation from cells: Room Temperature: Unacceptable, Refrigerated: 6 days, Frozen: Unacceptable.			

GENERAL INFORMATION	
Expected TAT	2 - 3 days
Notes	Labcorp Test Code: 884318
CPT Code(s)	83704
Service Provided By	 labcorp Oklahoma, Inc.