


**Chromium, Urine**

Order Name: **Chromi U Rand**  
Test Number: 5194545  
Revision Date: 12/12/2022

TEST NAME	METHODOLOGY	LOINC CODE
Chromium, Urine		

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Instructions	<p><b>Specimen Type:</b> Plastic urine container, no preservative</p> <p><b>Specimen Storage:</b> Maintain specimen at room temperature.</p> <p><b>Specimen Collection:</b> Sampling time is during shift or at the end of shift at the end of the work week for industrial exposure monitoring. For metals with multiphase elimination, the timing is given in relation to the work day exposure (shift) as well as to the work week exposure.</p> <p><b>Special Instructions:</b> Do not use preservative. Preservatives used for routine analysis may contain mercuric oxide (ie, Stabilur), which interferes with all metal testing. If both urinalysis and metal testing are ordered, please submit a separate urine specimen (containing no a</p> <p><b>Specimen Stability:</b> Ambient: 14 days, Refrigerated : 14 days, Frozen: 14 days</p>			

GENERAL INFORMATION	
Notes	Labcorp Test Code: 071530
CPT Code(s)	82495, 82570
Service Provided By	 <b>labcorp</b> Oklahoma, Inc.