

Chlamydia/Gonococcus/Mycoplasma genitalium, NAA, Urine

Order Name: **Ct Ng M gen UR**
 Test Number: 5195319
 Revision Date: 04/27/2026

TEST NAME	METHODOLOGY	LOINC CODE
Chlamydia/Gonococcus/Mycoplasma genitalium, NAA, Urine	<u>Nucleic acid amplification (NAA)</u>	

SPECIMEN REQUIREMENTS

Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	30 mL (2 mL)	Urine (See Inst.)	APTIMA Urine Transport Tube	Room Temperature
Instructions	<p>Specimen: First-void urine in Aptima® urine transport Volume: 2 mL of a 20 to 30 mL urine collection Container: Aptima® urine transport Collection: Urine specimen: The patient should not have urinated for at least one hour prior to specimen collection. Direct patient to provide a first-catch urine (approximately 20 to 30 mL of the initial urine stream) into a urine collection cup free of any preservatives. Collection of larger volumes of urine may result in specimen dilution that may reduce test sensitivity; lesser volumes may not adequately rinse organisms into the specimen. Female patients should not cleanse the labial area prior to providing the specimen. Add urine to the Aptima® Combo 2 urine collection device. The final volume must be between the two black lines on the device (about 2 mL). Storage Instructions: Maintain specimen at room temperature or refrigerate (2°C to 30°C). Stability Room temperature 30 days (stability provided by manufacturer or literature reference)</p>			

GENERAL INFORMATION

Expected TAT	3-4 days														
Notes	<table border="1"> <thead> <tr> <th>RESULT CODE</th> <th>RESULT NAME</th> <th>LOINC</th> </tr> </thead> <tbody> <tr> <td>6907667</td> <td>M genitalia Ur</td> <td>88226-6</td> </tr> <tr> <td>6910151</td> <td>Chlamydia Ur</td> <td>42931-6</td> </tr> <tr> <td>6910155</td> <td>Gonorrhea Ur</td> <td>60256-5</td> </tr> </tbody> </table>			RESULT CODE	RESULT NAME	LOINC	6907667	M genitalia Ur	88226-6	6910151	Chlamydia Ur	42931-6	6910155	Gonorrhea Ur	60256-5
	RESULT CODE	RESULT NAME	LOINC												
	6907667	M genitalia Ur	88226-6												
	6910151	Chlamydia Ur	42931-6												
6910155	Gonorrhea Ur	60256-5													
CPT Code(s)	87591, 87563, 87491														
Service Provided By															