

Pneumococcal Ab Pnl PCV20, 20 Serotypes

Order Name: **Pneumo 20**

Test Number: 5197339

Revision Date: 03/16/2026

TEST NAME	METHODOLOGY	LOINC CODE
Pneumococcal Ab Pnl PCV20, 20 Serotypes	Multiplex fluorescent immunoassay	See Below

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL (0.5 mL)	Serum	Clot Activator (SST or Red No-Gel)	Refrigerated
Instructions	Specimen: 1mL(0.5mL) Serum from SST clot tube Blood should be allowed to clot 30-60 minutes, then poured off into a plastic transport tube Storage Instructions: Refrigerated Stability Requirements: Ambient 6 weeks, Refrigerated 6 weeks, Frozen 6 weeks Causes for Rejection: Insufficient sample volume.			

GENERAL INFORMATION

Expected TAT	7-10 Days																																																																																				
Clinical Use	Pneumococcal antibody panel provides quantitative measurement of serotype-specific IgG antibodies to assess immune response to pneumococcal vaccination, particularly in immunocompromised patients.																																																																																				
Notes	<table><tr><th>RESULT CODE</th><th>RESULT NAME</th><th>RESULT LONG NAME</th><th>LOINC CODE</th></tr><tr><td>5575653</td><td>Pneumo 1 IgG</td><td>Pneumococcal Serotype 1 IgG</td><td>85955-3</td></tr><tr><td>5575659</td><td>Pneumo 3 IgG</td><td>Pneumococcal Serotype 3 IgG</td><td>86081-7</td></tr><tr><td>5197340</td><td>Pneumo 6A IgG</td><td>Pneumococcal Serotype 6A IgG</td><td>n/a</td></tr><tr><td>5575661</td><td>Pneumo 4 IgG</td><td>Pneumococcal Serotype 4* IgG</td><td>86108-8</td></tr><tr><td>5575663</td><td>Pneumo 5 IgG</td><td>Pneumococcal Serotype 5 IgG</td><td>86129-4</td></tr><tr><td>5575689</td><td>Pneumo 6B IgG</td><td>Pneumococcal Serotype 6B* IgG</td><td>40905-2</td></tr><tr><td>5575697</td><td>Pneumo 7F IgG</td><td>Pneumococcal Serotype 7F IgG</td><td>40911-0</td></tr><tr><td>5575667</td><td>Pneumo 8 IgG</td><td>Pneumococcal Serotype 8 IgG</td><td>86148-4</td></tr><tr><td>5575707</td><td>Pneumo 9V IgG</td><td>Pneumococcal Serotype 9V*, IgG</td><td>40926-8</td></tr><tr><td>5575691</td><td>Pneumo 10A IgG</td><td>Pneumococcal Serotype 10A IgG</td><td>86099-9</td></tr><tr><td>5575693</td><td>Pneumo 11A IgG</td><td>Pneumococcal Serotype 11A IgG</td><td>86123-7</td></tr><tr><td>5575671</td><td>Pneumo 12F IgG</td><td>Pneumococcal Serotype 12F IgG</td><td>85974-4</td></tr><tr><td>5575673</td><td>Pneumo 14 IgG</td><td>Pneumococcal Serotype 14* IgG</td><td>85992-6</td></tr><tr><td>5575699</td><td>Pneumo 15B IgG</td><td>Pneumococcal Serotype 15B IgG</td><td>47356-1</td></tr><tr><td>5575701</td><td>Pneumo 18C IgG</td><td>Pneumococcal Serotype 18C* IgG</td><td>40913-6</td></tr><tr><td>5575703</td><td>Pneumo 19A IgG</td><td>Pneumococcal Serotype 19A IgG</td><td>40915-1</td></tr><tr><td>5575679</td><td>Pneumo 19F IgG</td><td>Pneumococcal Serotype 19F* IgG</td><td>86021-3</td></tr><tr><td>5575683</td><td>Pneumo 22F IgG</td><td>Pneumococcal Serotype 22F IgG</td><td>86053-6</td></tr><tr><td>5575687</td><td>Pneumo 23F IgG</td><td>Pneumococcal Serotype 23F* IgG</td><td>86061-9</td></tr><tr><td>5575709</td><td>Pneumo 33F IgG</td><td>Pneumococcal Serotype 33F IgG</td><td>47349-6</td></tr></table>	RESULT CODE	RESULT NAME	RESULT LONG NAME	LOINC CODE	5575653	Pneumo 1 IgG	Pneumococcal Serotype 1 IgG	85955-3	5575659	Pneumo 3 IgG	Pneumococcal Serotype 3 IgG	86081-7	5197340	Pneumo 6A IgG	Pneumococcal Serotype 6A IgG	n/a	5575661	Pneumo 4 IgG	Pneumococcal Serotype 4* IgG	86108-8	5575663	Pneumo 5 IgG	Pneumococcal Serotype 5 IgG	86129-4	5575689	Pneumo 6B IgG	Pneumococcal Serotype 6B* IgG	40905-2	5575697	Pneumo 7F IgG	Pneumococcal Serotype 7F IgG	40911-0	5575667	Pneumo 8 IgG	Pneumococcal Serotype 8 IgG	86148-4	5575707	Pneumo 9V IgG	Pneumococcal Serotype 9V*, IgG	40926-8	5575691	Pneumo 10A IgG	Pneumococcal Serotype 10A IgG	86099-9	5575693	Pneumo 11A IgG	Pneumococcal Serotype 11A IgG	86123-7	5575671	Pneumo 12F IgG	Pneumococcal Serotype 12F IgG	85974-4	5575673	Pneumo 14 IgG	Pneumococcal Serotype 14* IgG	85992-6	5575699	Pneumo 15B IgG	Pneumococcal Serotype 15B IgG	47356-1	5575701	Pneumo 18C IgG	Pneumococcal Serotype 18C* IgG	40913-6	5575703	Pneumo 19A IgG	Pneumococcal Serotype 19A IgG	40915-1	5575679	Pneumo 19F IgG	Pneumococcal Serotype 19F* IgG	86021-3	5575683	Pneumo 22F IgG	Pneumococcal Serotype 22F IgG	86053-6	5575687	Pneumo 23F IgG	Pneumococcal Serotype 23F* IgG	86061-9	5575709	Pneumo 33F IgG	Pneumococcal Serotype 33F IgG	47349-6
RESULT CODE	RESULT NAME	RESULT LONG NAME	LOINC CODE																																																																																		
5575653	Pneumo 1 IgG	Pneumococcal Serotype 1 IgG	85955-3																																																																																		
5575659	Pneumo 3 IgG	Pneumococcal Serotype 3 IgG	86081-7																																																																																		
5197340	Pneumo 6A IgG	Pneumococcal Serotype 6A IgG	n/a																																																																																		
5575661	Pneumo 4 IgG	Pneumococcal Serotype 4* IgG	86108-8																																																																																		
5575663	Pneumo 5 IgG	Pneumococcal Serotype 5 IgG	86129-4																																																																																		
5575689	Pneumo 6B IgG	Pneumococcal Serotype 6B* IgG	40905-2																																																																																		
5575697	Pneumo 7F IgG	Pneumococcal Serotype 7F IgG	40911-0																																																																																		
5575667	Pneumo 8 IgG	Pneumococcal Serotype 8 IgG	86148-4																																																																																		
5575707	Pneumo 9V IgG	Pneumococcal Serotype 9V*, IgG	40926-8																																																																																		
5575691	Pneumo 10A IgG	Pneumococcal Serotype 10A IgG	86099-9																																																																																		
5575693	Pneumo 11A IgG	Pneumococcal Serotype 11A IgG	86123-7																																																																																		
5575671	Pneumo 12F IgG	Pneumococcal Serotype 12F IgG	85974-4																																																																																		
5575673	Pneumo 14 IgG	Pneumococcal Serotype 14* IgG	85992-6																																																																																		
5575699	Pneumo 15B IgG	Pneumococcal Serotype 15B IgG	47356-1																																																																																		
5575701	Pneumo 18C IgG	Pneumococcal Serotype 18C* IgG	40913-6																																																																																		
5575703	Pneumo 19A IgG	Pneumococcal Serotype 19A IgG	40915-1																																																																																		
5575679	Pneumo 19F IgG	Pneumococcal Serotype 19F* IgG	86021-3																																																																																		
5575683	Pneumo 22F IgG	Pneumococcal Serotype 22F IgG	86053-6																																																																																		
5575687	Pneumo 23F IgG	Pneumococcal Serotype 23F* IgG	86061-9																																																																																		
5575709	Pneumo 33F IgG	Pneumococcal Serotype 33F IgG	47349-6																																																																																		
CPT Code(s)	86581																																																																																				
Service Provided By	 labcorp Oklahoma, Inc.																																																																																				